

SteriHeel®

Heel Incision Safety Lancet

- **User safety:** blade is fully shielded before and after use. Activation button secured by safety catch.
- **User convenience:** pre-loaded, wider button activated design for easy operation.
- **Patient safety:** single-use only; cannot be re-used;sterilized by gamma irradiation; controlled penetration depth.
- **High blood flow:** controlled movement of the blade is designed to optimize blood flow.
- **Application:** primarily newborns and preemies. Also be recommended for other applications where short penetration depth and/or minimal compression of tissue is important.

Directions for Use

Please read the instructions carefully before use. Do not use SteriHeel if the sealed blister pack has been broken.

For more detail please visit www.steriheel.com



How to Use

- 


1 Clean the incision site with alcohol preparation pads (70% isopropanol) and allow the skin to air dry.
- 

2 Remove the safety trigger lock from the device.
- 

3 Raise the baby's foot to a suitable level and place the blade slot flush against the infant's heel site.
- 

4 When both ends of the device's flat face have made light contact with the skin, press the trigger and immediately move the device away from the heel.
- 

5 The first drop of blood should be wiped away gently with a dry sterile gauze pad.

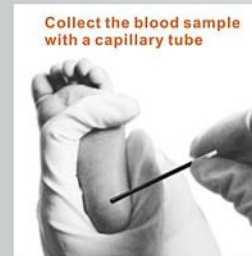

Collect the blood sample with a capillary tube

Collect the blood sample with a capillary tube (or similar device) and fill the blood collection container until the required volume is obtained. Gently press a dry sterile gauze pad on the incision site until the bleeding stops.



Collect the blood sample with a specimen collection card

Lightly press and allow the blood to drip on the specimen collection card directly.

Recorder #	Description	Color	Specification	Application	Package
05-036514	SteriHeel	Green	Low Flow 0.65mm Depth/1.4mm Incision Length	Micro-Preemie	50pcs/box 200pcs/case
05-038575	SteriHeel	Orange	Medium Flow 0.85mm Depth/1.75mm Incision Length	Preemie	
05-031025	SteriHeel	Yellow	Medium Flow 1.0mm Depth/2.5mm Incision Length	Newborn	
05-031428	SteriHeel	Pink	High Flow 1.14mm Depth/2.8mm Incision Length	Newborn	